GIMA

LARINGOSCOPI A FIBRE OTTICHE
FIBRE-OPTIC LARYNGOSCOPES
LARYNGOSCOPES À FIBRES OPTIQUES
LARYNGOSKOP MIT FIBER OPTIK LICHTLEITER
LARINGOSCOPIOS DE FIBRAS ÓPTICAS
LARINGOSCÓPIO A FIBRAS ÓPTICAS
ΛΑΡΥΓΓΟΣΚΟΠΙΑ ΜΕ ΟΠΤΙΚΕΣ ΙΝΕΣ
منظار الحنجرة العامل بالألياف البصريّة

MANUALE D'USO E MANUTENZIONE
USE AND MAINTENANCE BOOK
INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN
BETRIEBS UND WARTUNGSANWEISUNGEN
MANUAL DE USO Y MANTENIMIENTO
MANUAL DE USO E MANUTENÇÃO
ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ
دليل الإستعمال والرعاية

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FEATURES

The GIMA laryngoscopes are useful medical appliances to examine the larynx and to execute tracheal intubations.

The Green GIMA line is made by anti-magnetic stainless steel which is highly resistant to corrosion and uses fibre-optic blades and handles compatible with all major producers (certified ISO 7376-3). Light is transmitted from the handle to the tip of the blade through a fibre-optic bundle that can be taken apart to be sterilized independently or to be replaced in case of malfunction.

The handle uses two alkaline batteries "C" type 1,5 volt or rechargeable batteries, not supplied with the item.

The product or its components cannot be used for purposes different from the one specified in the present manual.

PRESCRIPTIONS

The product shall be used exclusively with alkaline batteries type "C" correctly installed. It shall not be dipped into water! When cleaning, make sure that the lamp socket or the battery compartment is not soaked with water.

Do not use the equipment in case it is damaged. Apply to your retailer. Avoid precarious repairs. Repairs shall be carried out with original spare parts only, which shall be installed according to the intended use.

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Disposal: The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.

For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.

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