Diabetes and/or skin insensitivity (Raynaud's Syndrome) of use on newborns, children and the elderly, in case of doubt about the treatment to be carried out and if the symptoms persist. **STORAGE:** Store out of the reach of children at room temperature, away from heat sources and flammable materials. **DISPOSAL:** Dispose of according to current regulations.

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EN INDICATORS: haematoma, sprain, injury, hyperthermia
and whenever cold application is required. WARNINGS and CONTRAINDICATIONS: not intended to come into direct contact with the skin; use with a cloth in between. Recommended treatment for approximately 20 minutes. For external use only, disposable. Do not freeze and do not use if package is damaged. Do not break the bag causing the contents to come out, which stain, irritate (in case of contact, rinse the area with water for 15 minutes) and are harmful if ingested (seek medical attention immediately). You must report any serious incident that occurs in relation to the medical device supplied by us, to the manufacturer and to the competent authority of the Member State in which you are located. Contact your doctor in case of cardiovascular, circulatory diseases

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Gima SpA