KONIX LIDO C STERİL KATETER JEL KULLANIM KILAVUZU / INSTRUCTION FOR USE OF KONIX LIDO C STERILE CATHETER GEL

ÜRÜN ADI: Konix Lido C Sterile Kateter Jel

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PRODUCT NAME: Konix Lido C Sterile Catheter Gel

DESCRIPTION: Konix Lido C Sterile Catheter Gel is a sterile water soluble, non-irritating lubricant gel with supportive antiseptic and anesthetic properties. It is used to facilitate the insertion of catheters and other medical instruments (catheterization, cystoscopy) into the urethra and bladder.

INGREDIENTS: 100 g of Konix Lido C Sterile Catheter Gel contains: Deionized Water, Hydroxyethyl Cellulose (Lubricative), Mono Propylene Glycol (Moisturizer), 2 g Lidocaine (Local anesthetic), 0.05 g Chlorhexidine Gluconate (Antiseptic), Methyl Hydroxybenzoate (Preservative), Propyl Hydroxybenzoate (Preservative)

APPLICATION:

1. Wash your hands and prepare an area that is aseptic as possible to perform the operation. Prepare a clean serving tray and place the syringe on it.
2. Remove the syringe (6 ml, 11 ml, 12 ml and 12.5 g) from its sterile package by tearing off 6 ml syringe is clean suitable for use in women and children, while 11 ml, 12 ml syringe and 12.5 g according syringe are suitable for use in men.
3. Lightly press on the plunger before removing the blue cap to provide more even and gentle application. For the according syringe, break off the applicator tip.
4. Carefully insert the cone of the syringe into the urethral opening and slowly inject the gel by applying gentle and even pressure.
5. Apply the remaining gel to the surface of the catheter or to the surface of another medical device.
6. Konix Lido C Sterile Catheter Gel shows lubricative effect as soon as the application starts. Its anesthetic effect starts 3-5 minutes later.

7. If more gel has to be used, condition of the applied tissue, age of the patient and any adverse condition of the patient which may be related to sensitivity of chlorhexidine must be taken into care by the physician.

8. When the whole product is not used, the syringe and the remaining gel should be evaluated and discarded like other medical wastes.

9. Konix Lido C Sterile Catheter Gel should only be used under the supervision of specialist health personnel.

CONTRAINDICATIONS: Konix Lido C Sterile Catheter Gel should not be used in patients with hypersensitivity to any of the components or patients with marked bradycardial rhythm disorders. Konix Lido C Sterile Catheter Gel should not be used in patients with severe cardiac insufficiency, severe hepatic and renal dysfunction, traumatic mucosa and / or inflammation / sepsis at the application site.

WARNINGS AND PRECAUTIONS FOR USE

- Under anesthesia a lubricant without lidocaine should be used.
- Installation should only be performed by a physician or by a qualified medical personnel.
- Patients with glucose-6-phosphate dehydrogenase deficiency or congenital or idiopathic metemoglobinemia are more susceptible to drug induced metemoglobinemia.
- When applying Konix Lido C Sterile Catheter Gel, a temporary burn may occur; patients should be warned in this regard. In this case, cold compress to the applied region can alleviate the burning sensation.
- Allergic reactions to lidocaine and / or chlorhexidine may rarely occur. Please consult your doctor if redness, burning and stinging occurs in the applied site.
- Irritates eyes and/or skin in case of direct contact. Wash with plenty of water in case of exposure.
- If swallowed, seek medical advice.
- Keep away from children.
- Store in dry and cool places.
- Keep away from direct sunlight.
- For single use only.
- Do not use products with damaged packages because of the risk of sterilization deterioration.

PREGNANCY AND LACTATION: Konix Lido C Sterile Catheter Gel can be used during pregnancy and lactation. The risk/benefit assessment should be performed by the physician. Lidocaine passes to placenta and enters breast milk in small quantities. Its use is contraindicated in pregnant women and infants.

DRUG INTERACTION

- Interaction due to absorption of lidocaine may occur when used in combination with the following drugs: Propranolol: Reduction in plasma clearance of lidocaine, Cimetidine: Reduction in plasma clearance of lidocaine, Antiarrhythmic products: Increase in toxicity of lidocaine, Phenitoin or barbiturates: Reduction in plasma levels of lidocaine. The specified interactions can be seen in long-term use and repeated high doses. No clinically significant interactions have been reported when administered at the recommended doses.
- Due to the risk of interaction depending on chlorhexidine, use with the following is not recommended: Chlorhexidine Gluconate (CHX) + Sodium Hypochlorite (NaOCl) Chlorhexidine Gluconate (CHX) + Hydrogen Peroxide (H2O2) Chlorhexidine Gluconate (CHX) + Povidone-Iodine (BTD)
- Contact to sterilized lidocaine + NaOCl interaction is known to produce toxic by-products for humans.
- SIDE EFFECTS:** Allergic reactions to lidocaine and chlorhexidine are extremely rare. In the case of hypersensitivity, skin reactions are possible. Skin reactions depending on anaphylaxis (hypersensitivity) may occur due to the use of lidocaine in the damaged mucosa. Systemic adverse reactions to lidocaine are rare and may result from hypersensitivity or diminished tolerance.

PACKAGING

- Konix Lido C Sterile Catheter Gel is supplied as sterile disposable packages, pre-filled in 6 ml (= 6 g), 11 ml (= 11 g), 12 ml (= 12 g) syringes and 12.5 g according syringe.
- Konix Lido C Sterile Catheter Gel 6 ml
- Konix Lido C Sterile Catheter Gel 11 ml
- Konix Lido C Sterile Catheter Gel 12 ml
- Konix Lido C Sterile Catheter Gel 12.5 g

STERILIZATION: Konix Lido C Sterile Catheter Gel is sterilized with radiation after the packaging. In case the package damaged, do not use since it will lose the sterility.

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